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Intended purpose:

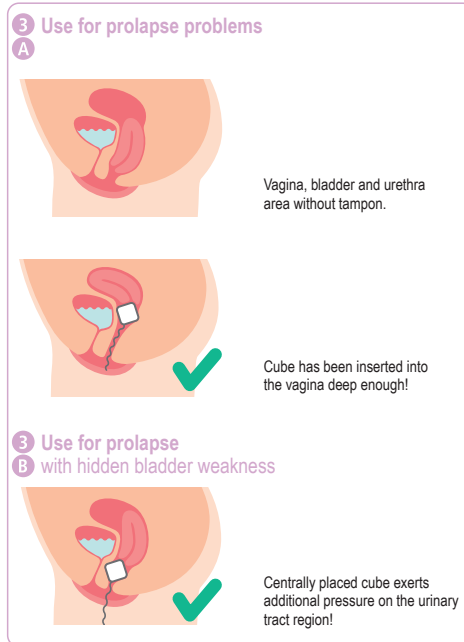
The cube-shaped Contam® vaginal tampon can be used to treat various degrees of vaginal and uterine prolapse as well as bladder emptying disorders.

Material data:

Tampon: PVA foam, string: cotton, applicator: polystyrene

Directions for use:

- 1 Wash hands thoroughly. Open the film along the perforated line, remove the tampon and unwind the removal string.
- 2 Soak Contam® briefly in warm water (approx. 1 minute), squeeze out remaining liquid. Contam® is soft and ready for use. Contam® is inserted in the same way as a menstrual tampon.

- 4 If it does not feel comfortable, please move the tampon into a comfortable position with your fingertips. You cannot feel Contam® when it is positioned correctly. The end of the retrieval string must be outside the vagina.

- 5 To remove Contam®, you should adopt a comfortable position. Find the best direction for removing the tampon by gently pulling on the retrieval string.

- 6 Dispose of Contam® in household waste; **do not throw it into the toilet**. Wash your hands afterwards.

Using the toilet: There is **no** need to remove the Contam® vaginal tampon when urinating.

Med SSE does not accept any liability for damages arising from non-adherence to the instructions for use.

Safety notes:

- You should change Contam® after swimming/bathing.
- In the unlikely event of the retrieval string breaking despite our careful monitoring through continuous inspection, press while squatting (like for a bowel movement), try to get hold of the tampon with two fingers and remove it. Should that be unsuccessful, go and see a doctor immediately.
- Contam® is a disposable product and must not be reused. Hygiene requirements are no longer met if the product is reused!
- Contam® must not remain inside the vagina for more than 12 hours. Patients who cannot ensure compliance with this requirement must not use Contam®.
- If you suddenly experience fever, diarrhoea, vomiting, bleeding, dizziness or skin irritation while using Contam®, you must remove the tampon immediately and see a doctor.
- Do not use Contam® after the expiration date. Keep away from direct sunlight and heat; store in a dry place.

Contraindications (do not use Contam®):

- PVA allergy
- menstruation
- while sleeping or during infections
- seek medical advice before use in pregnancy

Side effects:

There are no known side effects

Every serious incident that occurs in relation to the medical device must be reported to the manufacturer, who will report this to the responsible authority.

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