Safety with comfort.

EN

Contam®

... for a safe feeling!

WEAK BLADDER/PROLAPSE No problem for you!

Bladder weakness is a very personal problem. We have developed tampons for its conservative treatment.

Intended purpose:

- Stress incontinence in women
- Prolapse
- Mixed incontinence with urge incontinence
- Recurrent incontinence after surgery
- Prevention in association with physical exercise (work-out)

Use and mode of action of Contam® for incontinence (weak bladder)

Vaginal area without tampon
Vagina, bladder and urinary tract area without tampon.

Correct use!
The tampon in the vagina exerts pressure on the urinary tract, thus preventing the uncontrolled discharge of urine. Put the tampon in a position that is comfortable for you.

Incorrect use!
The tampon has been inserted too deep into the vagina!

Directions and instructions for use:

1. Wash your hands thoroughly prior to use.
2. Open the protective film along the perforated line, remove the tampon and unwind the removal string.

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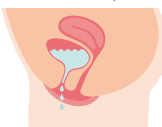
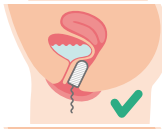
3. Soak Contam® briefly in warm water (approx. 1 minute).
4. Squeeze out remaining liquid. Contam® becomes soft and elastic and is ready for use.
5. Contam® is inserted in the same way as a menstrual tampon. Adopt a comfortable position for inserting, e.g. by placing a leg on a stool or sitting on the edge of the toilet lid.
- 6A. **Use for prolapse problems**
With your fingertips, insert Contam® deep into your vagina. Then pull it back slightly so that you can touch it with your fingertips
- 6B. **Use for prolapse problems with hidden bladder weakness**
With your fingertips, insert Contam® as deeply as possible into your vagina. Note: The tampon cannot slip away inside your body!
- 6C. **Using Contam® with an internal hole**
Place the tampon with the internal hole onto the spike of the applicator and then insert the tampon.
7. If it does not feel comfortable, please move the tampon into a comfortable position with your fingertips. There are few nerves in the middle part of the vagina. For that reason, you cannot feel the correctly positioned Contam®.
8. The end of the retrieval thread must be outside the vagina.
9. To remove Contam®, you should also adopt a comfortable position. Find out the best direction for removing the tampon by pulling gently on the removal thread.
10. Place the used Contam® into a small bag and dispose of it in normal household waste. Please wash your hands after removing and disposing of Contam®.

Using the toilet: There is no need to remove the Contam® vaginal tampon when urinating.

Safety notes:

- You should change Contam® after swimming/bathing.
- In the unlikely event of the retrieval thread breaking despite our careful control through continuous monitoring, press while squatting (like for a bowel movement), try to get hold of the tampon with two fingers and remove it. Should that be unsuccessful, go and see a doctor immediately
- Contam® is a disposable product and must not be reused. Hygiene requirements are no longer met if the product is reused!
- Contam® must not remain inside the vagina for longer than 12 hours. Patients who cannot ensure compliance with this requirement must not use Contam®
- If you suddenly experience fever, diarrhoea, vomiting, bleeding, dizziness or skin irritation while using Contam®, you must remove the tampon immediately and see a doctor

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Contraindications (do not use Contam®):

- PVA allergy
- Menstruation
- While sleeping or during infections
- Seek medical advice before use in pregnancy

Side effects: There are no known side effects

Labelling of medical devices:

The packaging and package inserts for Contam® vaginal tampons contain pictograms that are explained in the following:

- CE Symbol for "Conformity with current European guidelines"
- ⊗ Symbol for "Not for reuse"
- i Symbol for "Read the instructions for use"
- ⚙ Symbol for "Manufacturer"
- ⌚ Symbol for "Use by"
- LOT Symbol for "Batch number"
- MD Symbol for "Medical device"
- ☀ Symbol for "Keep away from heat"
- ☔ Symbol for "Keep dry"

Product code 15.25.21.2004

Material data: Tampon: PVA foam, Thread: Cotton, Applicator: Polystyrene

Med SSE does not accept any liability for damages arising from non-adherence to the instructions for use.

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GmbH